



forest dental **australia**

affordable excellence + designer friendly

SELF-CONTAINED WATER BOTTLE QUICK DISCONNECT UPGRADE KIT


INSTALLATION INSTRUCTIONS

Contents:

1 x 1.5 litre water bottle with bottle adapter attached

1 x quick disconnect converter

1 x water tube

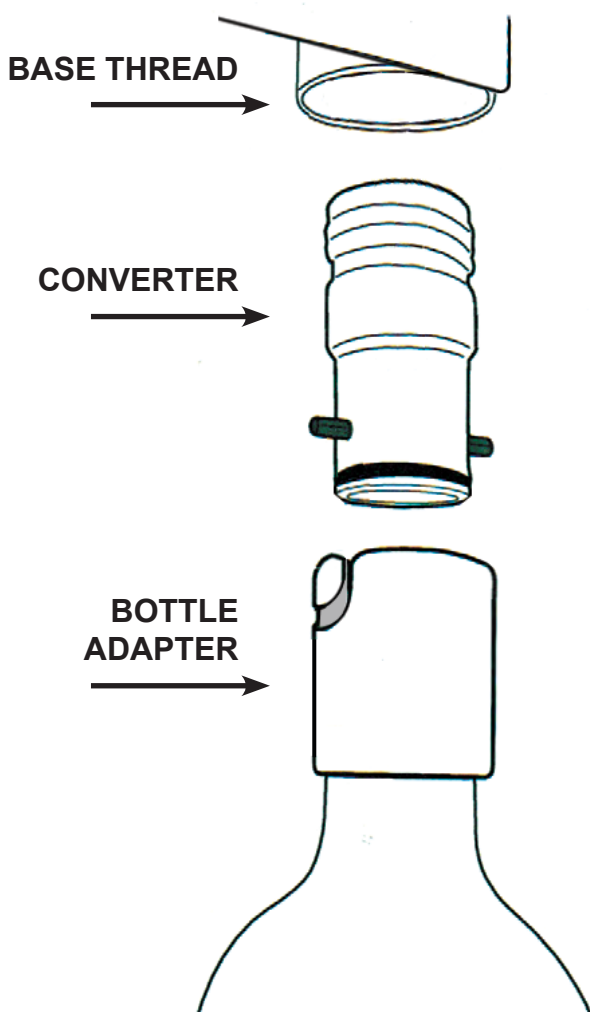

INSTRUCTIONS

- Remove the water bottle from its base thread.
- Remove the existing water tube from its barb (the tube may have to be cut off the barb) and attach the new longer tube.
- Firmly screw the converter into the base thread (where the bottle originally was).
- The bottle can then be connected to the converter using a bayonet action.

For additional bottle adapters, order Part No. 1100-422.

For 1.5 litre bottles with adapters fitted, order Part No. 1100-431.

PO Box 126 Blackheath, Beautiful Blue Mountains, NSW 2785 Australia

Phone: +612 4787 5382 • Fax: +612 4787 7469

Email: info@forestdental.com.au • Web: www.forestdental.com.au